FR HEMOROBEE® pommade

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GB HEMOROBEE® ointment

FOR EASING THE DISCOMFORT CAUSED BY THE HEMORRHOIDS

Medical device

Ingredients: PETROLATUM, LANOLIN CERA, PROPOLIS EXTRACT, ANTHEMIS NOBILIS.

Intended use

HEMOROBEE® ointment provides protective film for decreasing the rectal mucosa irritation and painful defecation. HEMOROBEE® ointment creates protective film on the damaged rectum mucosa and thus facilitates the unpleasant symptoms caused by hemorrhoids (flatulence, pain, burning, itching), which are usually the result of secondary inflammation in the anal area. The resulting film prevents contact between the stool and damaged mucosa. With facilitated defecation (without pain and strain), mucosa recovers very quickly.

Instructions for use

During the acute periods, apply HEMOROBEE® ointment on clean anorectal area in the morning and in the evening. Later, when symptoms relieve apply only in the evening. It can also be applied by using applicator put on the tube instead of the cap. Before use, remove the protector from the applicator's top. Right after using the applicator, remove it from the tube which should be closed by the cap. Thoroughly clean the applicator with water, dry and close it with the protector. If the ointment is applied with fingers, take care of hand hygiene and apply the ointment with fingertips.

Contraindications

Allergic reaction caused by hypersensitivity to an ingredient of the product.

Possible side effects

There have been no side effects reported while using Hemorobee ointment. In the case of hypersensitivity to one of the components of Hemorobee ointment, it may occur reaction in form of redness, rash and itching of the skin. If you notice any of these symptoms described here, consult your doctor or pharmacist. If you notice any others symptoms, consult also your doctor or pharmacist.

Persons allergic to propolis should not use this product! Keep out of the reach of children.

Safekeeping

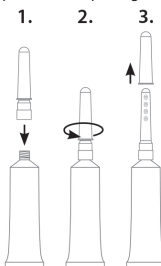
Store in a dry and cool place. The product can be used for 2 months after opening.

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